

# Combo Swab

01




**Wear disposable polygloves.**

02



**Open only half of the package and place it on the table floor.**

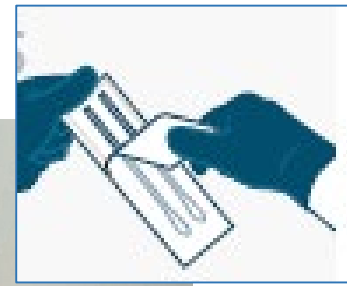

03



**Leave the sterile tube open.**

04



**While wearing a mask, blow your nose twice and cough twice.**

## Combo Swab 채취 방법

05



Grab the handle of one of the two swabs and take it out.

06



Slowly rotate the swab, gently pressing against the inside of your nostril at least 5 times.

07



Place the swab in the sterile tube and snap off the end of the swab at the break line, so that it fits comfortably in the tube.

08



Place the cap on the tube slightly.

## Combo Swab 채취 방법

09



Grab the handle of the other swap and take it out.

10



Rub the four gums on the molars and the lower part of the tongue smoothly.

11



Place the swab in the sterile tube and snap off the end of the swab at the break line.

12



After the tube cap is completely closed, stick it in the tube rack.